

mypak SOLUTIONS

software consumables automation

TriFold Packing Guide


TriFold Pack Components




TriFold Card



TriFold Label

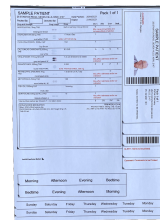

Blister size of your choice
(5 depth options)

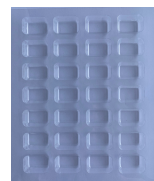

Platen




Roller




Step 1

Print the TriFold label from the myPak software

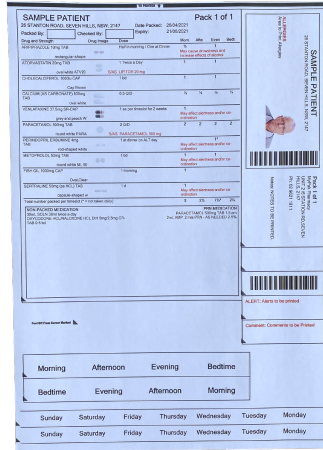

Step 2

Peel back the plastic backing to expose the glue on right hand side of the TriFold card. Leave the middle section intact.

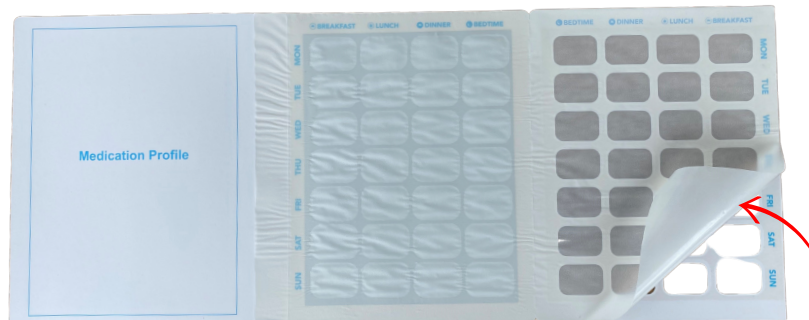

Step 3

Align the TriFold card cut-outs over the Platen, then carefully insert the blister size of choice into TriFold card



Step 4- remove

Apply the roller over the blister and card to ensure best adhesion



Step 5

Pack medication according to the patient profile displayed on the TriFold label or from the guided fill screen via the myPak software



Step 6

Once all of the medication has been packed, peel back remaining backing to expose glue on the card (middle section). Fold the exposed middle section to the seal pack



Step 7

Use the roller to ensure best adhesion to the pack



Step 8

Apply the labels as directed below



Completed Pack

